

InstantCHEK HIV 1+2



InstantCHEK - HIV 1+2



- **Rapid test - results in less 1 minute**
- **High Sensitivity (100%) & Specificity (>99.5%)**
- **3 step test - Easy to use and interpret**
- **Can be stored at room temperature**
- **Instruction manual available in English. Other languages upon request.**

InstantCHEK – HIV 1+2 is used for the rapid and simple screening of antibodies to the two human immunodeficiency viruses, HIV-1 and HIV-2 in human serum, plasma or whole blood. The test is intended for preliminary screening in emergency situations such as in the field or in transplant procedures. **InstantCHEK HIV 1+2** can also be used by small clinics, reference laboratories or blood transfusion centers as a second screen when two screening methods are required on each reactive sample. Results should be confirmed through other testing means.

The assay requires no specialized laboratory equipment and the kits can be stored at room temperature. Kits include all necessary reagents, controls and materials to perform the number of tests indicated per kit size.

KIT SIZE

- 40 tests per kit or
- 100 test per kit.

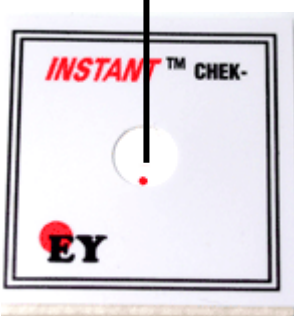
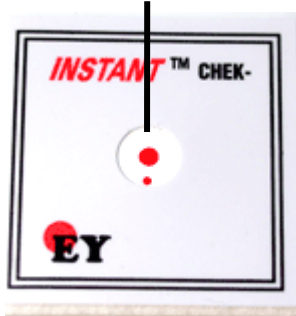
**Do you have any question or would you be interested to receive our quotation?
Please contact us by e-mail (sales@helveticahealthcare.com)
or by phone (+41 22 534 94 54)**

EASY USE AND INTERPRETATION

To run the *Instant*CHEK HIV 1+2 device:

1. Add 1 drop (or 40 ul) of undiluted SAMPLE and allow to soak in.
2. Add 2 drops (or 80ul) of LIQUID GOLD and allow to soak in.
Proceed to step 3 immediately
3. Add 6 drops of WASH BUFFER and allow to soak in.
Read result immediately

*Instant*CHEK HIV 1+2 gives a clear result for positive samples (two visible spots) and a clear result for negative samples (one visible spot). *Instant*CHEK HIV 1+2 includes a built in assay control. The appearance of one spot for negative samples gives an added measure of quality control by demonstrating that the assay procedure was done correctly and that the reagents are chemically active.

Control spot	Test spot	Device Pattern After Assay	Interpretation
 <p>NEGATIVE</p>	 <p>POSITIVE</p>	NO VISIBLE RED / PINK SPOT in test area but PINK / RED CONTROL SPOT visible.	NEGATIVE
		Distinct RED / PINK SPOT in test area and RED / PINK CONTROL SPOT present.	POSITIVE
		No VISIBLE RED / PINK SPOT in test area and NO RED / PINK CONTROL SPOT visible.	INVALID

External Evaluations

A copy of the WHO evaluation of InstantCHECK HIV 1+2 is available from Helvetica Health Care upon request and can also be found on our web site.

**Do you have any question or would you be interested to receive our quotation?
Please contact us by e-mail (sales@helveticahealthcare.com)
or by phone (+41 22 534 94 54)**